Instructions for Use

BISCO, Inc.
1100 W. Irving Park Road
Schaumburg, IL 60193
U.S.A.
1-800-247-3368
1-847-534-6000
www.bisco.com

BISICO France
208, allée de la Coudoulette
13680 Lançon de Provence
France
33-4-90-42-92-92

IN-176R10

Rev. 10/20

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.

en **CORE-FLO™ DC**
Dual-Cured Flowable Core
Build-Up Composite

GENERAL INFORMATION

CORE-FLO DC is a flowable dual-cured, radiopaque core build-up composite. **CORE-FLO DC** is specifically designed to cement posts and build-up the core structure in one easy application. **CORE-FLO DC** is also an ideal dentin replacement material. **CORE-FLO DC** comes in an automix dual-syringe and is dispensed with the DUAL-SYRINGE DISPENSER. **CORE-FLO DC** is intended for use with a bonding agent that is compatible with dual-cured materials, such as BISCO adhesives.

Indications for use:

The principle indications for use of **CORE-FLO DC** are:

1. Core build-up material
2. Cementation of posts or pins
3. Dentin replacement material

Intended user: licensed dentist

Warnings:

- Contamination by saliva, blood, or water should be avoided during these procedures. When using dental adhesives, contamination will compromise dentin bonding and may result in decreased longevity of the restoration.
- A rubber dam is recommended for all indications where contamination may be possible

Cautions:

Cross-contamination:

- Product may contain items that are designed for one-time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
- Use of commonly available hygienic protective covering to avoid any contamination following during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.

Precautions:

- Avoid contact with the skin; unpolymerized (meth) acrylate resins may cause skin sensitization in susceptible persons. In case of contact, wash skin with soap and water.
- To prevent polymerization or clogging, the dual-syringe mixing tip should be left in place until the next application. If reused introrally, wipe mixing tip with disinfectant prior to storage.
- Refer to the manufacturer's instructions. Unlike BISCO adhesives, not all one bottle total-cure or self-cure systems bond to self-cured or dual-cured materials.
- To isolate the core build-up for better impressions or fabricating temporary crowns/bridges (it may be sufficient to moisten it thoroughly), in order to prevent any bonding with the teeth.
- Refer to DUAL-SYRINGE DISPENSER INSTRUCTIONS FOR USE for proper use, handling, and care.
- If refrigerated, allow **CORE-FLO DC** to reach room temperature before use.
- Individual component labels for specific expiration dates.
- Safety data sheet available on request.
- Safety data sheet available at www.bisco.com.
- Instructions for Use available upon request.
- Instructions for Use available at www.bisco.com.

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